

Assistive Technology Training Online

University at Buffalo - The State University of New York
USDE# H324M980014
<http://at-training.com>



IntelliKeys Switch Settings Chart

Level #	Switch #1 (Left Side)	Switch #2 (Right Side)
1 - Apple default	Open Apple (⌘)	Closed Apple (Option)
1 - PC default	Spacebar	Enter
2	Spacebar	Return/Enter
3	Down Arrow	Return/Enter
4	Right Arrow	Return/Enter
5	Tab	Return/Enter
6	Escape	Return/Enter
7	Left Arrow	Right Arrow
8	Up Arrow	Down Arrow
9	1	2
10	Left Shift	Right Shift
11	Click	Double Click

IntelliKeys USB Switch Settings Chart

Level #	Switch #1 (Left Side)	Switch #2 (Right Side)
1	Control-Enter/Return	Control-Enter/Return
2	Control-Right Arrow	Control-Down Arrow
3	Space	Enter/Return
4	Click	Click
5	Click	Double Click
6	Left Arrow	Right Arrow
7	Up Arrow	Down Arrow
8	Left Arrow	Up Arrow
9	1	2
10	Tab	Enter/Return
11	Tab	Space
12	Tab	Shift-Tab
13	Left Shift	Right Shift
14	Down Arrow	Enter/return
15	Right Arrow	Enter/return

Choosing Switch settings

IntelliKeys

- Slide the Setup Overlay onto IntelliKeys
- Press KEYBOARD RESET twice. You will see the lights on IntelliKeys flash, and hear a beep. This step is necessary to remove any custom overlay currently in the IntelliKeys memory.
- Press the Custom Overlay Level key.
- Press a number between 1 and 11 on the Number Pad. For example, to select Level 11 (the mouse click level), press the 11 key. Be sure to press "11" instead of "1" "1".
- **Remove the Setup Overlay.**
- The switches have been configured; plug them in and use them.
- In addition to switch use, the left side of the keyboard acts as Switch #1 and the Right side as switch #2.

In Level 11 where the switches act as mouse clicks requires that Mouse Keys are turned on in the control panel. See your owner's manual.

IntelliKeys USB

- Slide the Setup Overlay onto IntelliKeys
- Press **Show Control Panel** in the Troubleshooting Section.
- Click the **Advanced** Tab. Select **As a switch, with these settings.**
- From the drop down list box (similar to the table above), select **switch Setting 5 - click and Double Click**. Click Done.
- **Remove the Setup Overlay**
- The switches have been configured; plug them in and use them.
- In addition to switch use, the left side of the keyboard acts as Switch #1 and the Right side as switch #2.